



# WOUND RULER

CUSTOMIZE THE LENGTH OF THE RULER TO THE SIZE OF THE WOUND

FIVE RULERS IN ONE

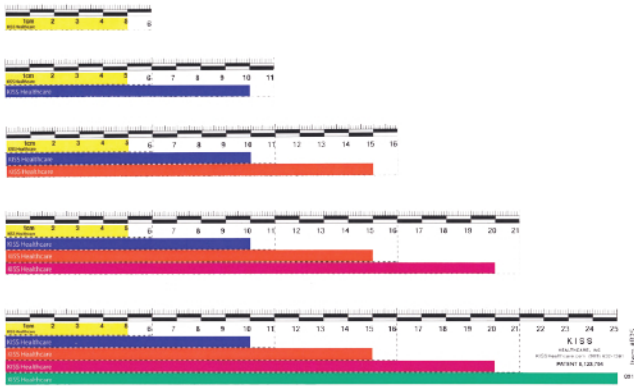


Select a ruler longer than the wound.

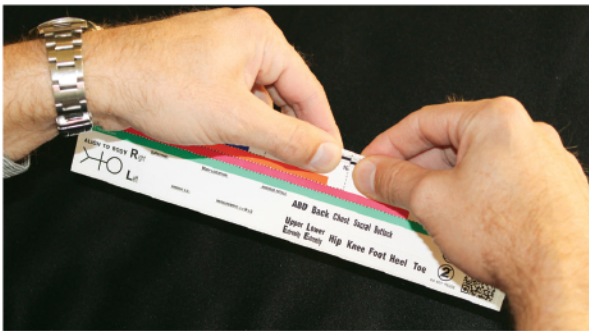


Tear the desired length along the perforated lines



Tear down the length, removing all the smaller bars.

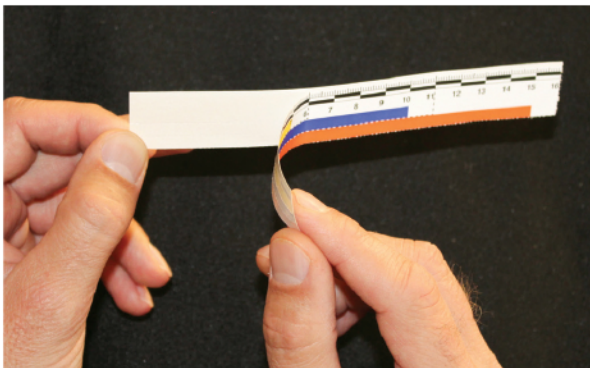

Remove the ruler from the backing

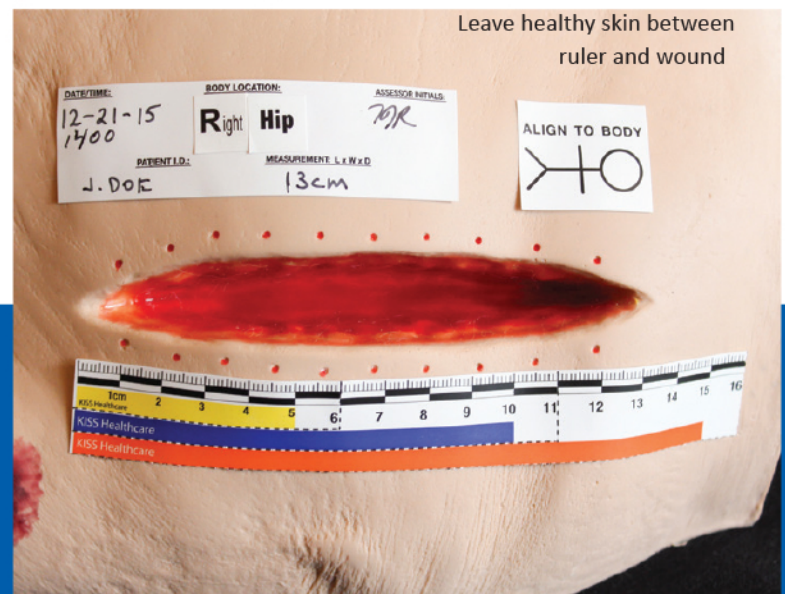

- Apply ruler next to the wound
- Apply the stick figure to align the wound
- Fill out the note label (ultra fine-tip permanent marker works best), and apply next to ruler
- Apply the right/left and location tabs to the note label

KISSsystem.com 909-632-1361